
Raiser Pro Heel Strap

Assembly instruction

molift[®]
a part of Etac

BM49099 Rev. A 2020-10-08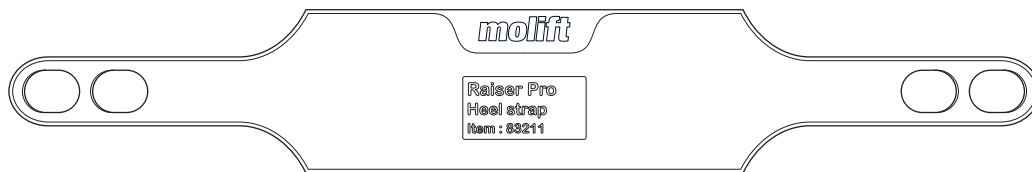

Contents

Assembly instruction - English	- 4
Monteringsanvisning - Svenska	- 6
Monteringsanvisning - Norsk	- 8
Monteringsvejledning - Dansk	- 10
Asennusohje - Suomi	- 12
Montageanleitungen - Deutsch	- 14
Montage-instructies - Nederlands	- 16
Instructions de montage - Français	- 18
Istruzioni di assemblaggio - Italiano	- 20
Instrucciones de ensamblaje - Español	- 22

Assembly instructions

83211

Raiser Pro HeelStrap

BM49001 2020 / 09 / 21

Read the Raiser Pro main User Manual before use!

It is important to fully understand the content of the main user manual before attempting to use the equipment. The main User Manual for Molift Raiser Pro contains important safety instructions and information regarding the use of the Molift Raiser Pro. Scan the QR code below to find the main user manual for Molift Raiser Pro and additional documentation.

Raiser Pro HeelStrap

Raiser Pro HeelStrap HeelStrap is an accessory to be used when the patient needs support to maintain the feet in the correct position when standing on the baseplate of the Raiser Pro. The product shall be mounted onto the Molift Raiser Pro turning knobs. The HeelStrap has two holes on each side and can be adjusted into 3 different length. The HeelStrap shall provide support against the user's heels.

Assess the risk and take notes.

You as the assistant are responsible for the safety of the user.

Revise the risk-assessment as often needed due to the patient's condition.

Actions to be performed daily or before use:


- Make sure the user has the required ability for the use of the HeelStrap
- Make sure the HeelStrap does not have visible damage or frays.

Material and cleaning instruction

Molift Raiser Pro HeelStrap is made out of Silicone.

Use 70 % alcohol or 45% isopropanol for cleaning and disinfection

Find out which combination to be used for each user.

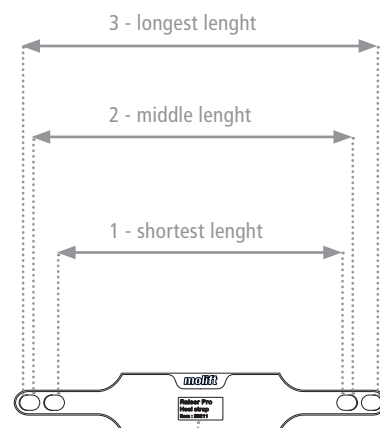Material recycling identification

Manufacturing year


Manufacturing month

The wheels must be locked before the heel strap is mounted onto the knobs

Mounting the Raiser Pro Heelstrap

Make sure the user is sitting down and that the wheels on the Molift Raiser Pro is locked before mounting the HeelStrap. Make sure the HeelStrap is correctly fastened before transferring the user.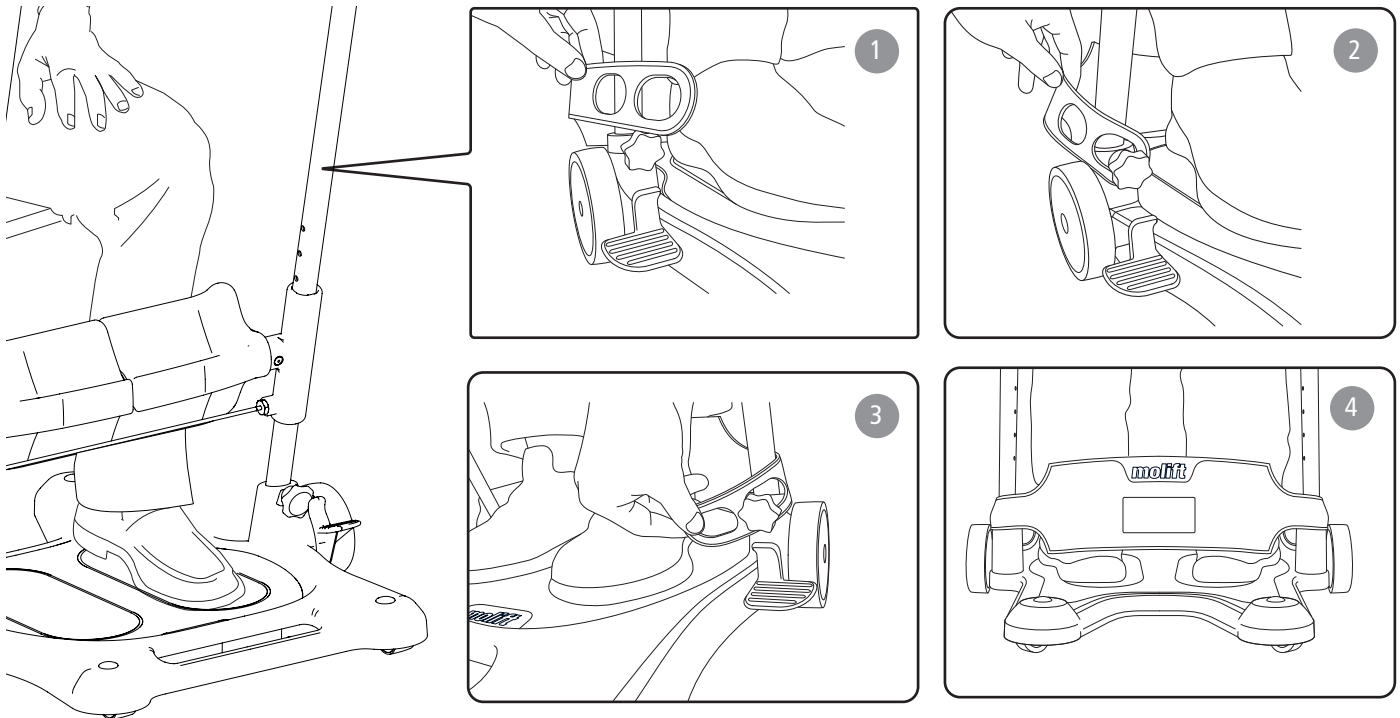

- 1 The user is sitting down, the feet are placed on the footplate and the height of the leg support is adjusted to the desired height as described in the Molift Raiser Pro user manual. Make sure the wheels are locked.
- 2 Fasten the HeelStrap by attaching one side to the turning knob. Make sure it is securely fastened before attaching on the other side.
- 3 Place the HeelStrap around the users heels and fasten the HeelStrap on the turning knob on the other side of the Molift Raiser Pro. The HeelStrap has two holes on each side and can be adjusted into 3 different length depending on the size and stance of the user's feet.
- 4 Make sure the HeelStrap is fitted firmly around the user's heels and as low as possible, preferably in contact with the Raiser Pro base plate. The HeelStrap shall provide support against the user's heels.

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The Etac logo is displayed on a solid grey rectangular background. The logo itself is white and features a stylized 'e' with a dot inside a circle, followed by the lowercase letters 'etac' in a bold, sans-serif font, with a registered trademark symbol (®) to the upper right.