
# Raiser Pro Block

Assembly instruction

**molift**<sup>®</sup>  
by Etac

BM52099 Rev. C 2021-08-12

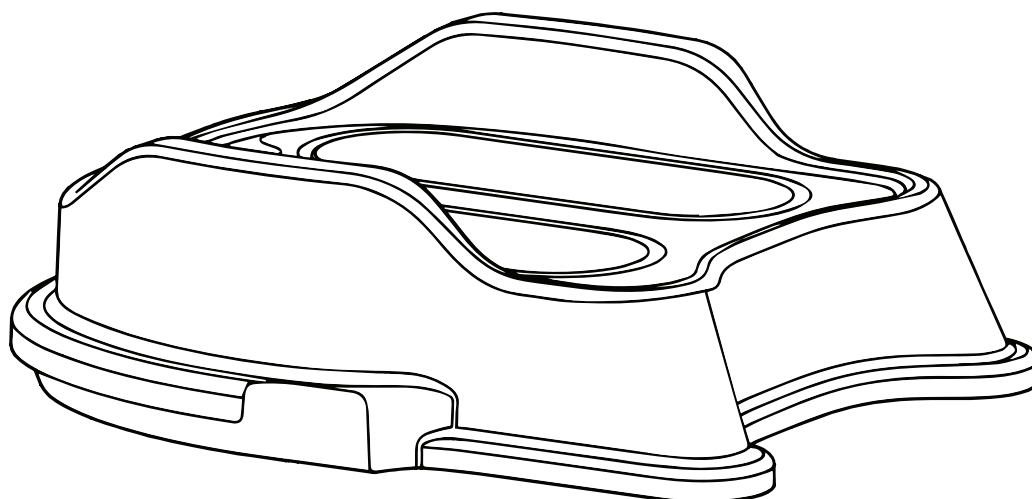## Contents

|  |
|--|
| Assembly instructions - English - 4        |
| Monteringsanvisning - Svenska - 6          |
| Monteringsanvisning - Norsk - 8            |
| Monteringsvejledning - Dansk - 10          |
| Asennusohje - Suomi - 12                   |
| Montageanleitungen - Deutsch - 14          |
| Montage-instructies - Nederlands - 16      |
| Instructions de montage - Français - 18    |
| Istruzioni di assemblaggio - Italiano - 20 |
| Instrucciones de ensamblaje - Español - 22 |

## Assembly instructions

# 28612

## Molift Raiser Pro Block

### Read the Raiser Pro main User Manual before use!

It is important to fully understand the content of the main user manual before attempting to use the equipment. The main User Manual for Molift Raiser Pro contains important safety instructions and information regarding the use of the Molift Raiser Pro. Scan the QR code below to find the main user manual for Molift Raiser Pro and additional documentation.

### Raiser Pro Block

Raiser Pro Block is an accessory to be used when the patient needs a higher foot position when using the Raiser Pro. It raises the height by 9cm. The product fits onto the Molift Raiser Pro base plate. The Raiser Pro Block does not need any kind of fixing, it fits directly onto the base plate.

### Safe Working Load (SWL):

170kg (374 lbs)

### Assess the risk and take notes.

You as the assistant are responsible for the safety of the user.

Revise the risk-assessment as often as needed due to the patient's condition.

### Actions to be performed daily or before use:



- Make sure the user has the required ability to use the Block
- Make sure the Block does not have visible cracks or other damage.

### Material and cleaning instruction

Molift Raiser Pro Block is made of Polypropylene.

Use 70% alcohol or 45% isopropanol for cleaning and disinfection

Information regarding, SWL, production date, material, name and art number can be found on the underside of the Raiser Block

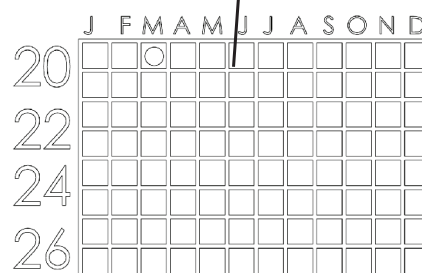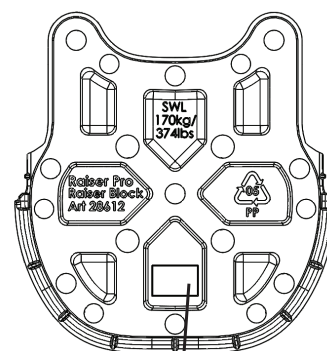Manufacturing month and year


Material recycling identification

Make sure the Raiser Block is correctly fitted before use.

## Mounting the Raiser Pro Block

Make sure the user is sitting down and that the wheels on the Molift Raiser Pro is locked before mounting the Raiser Pro Block. Make sure the Raiser Pro Block is correctly fitted before transferring the user.

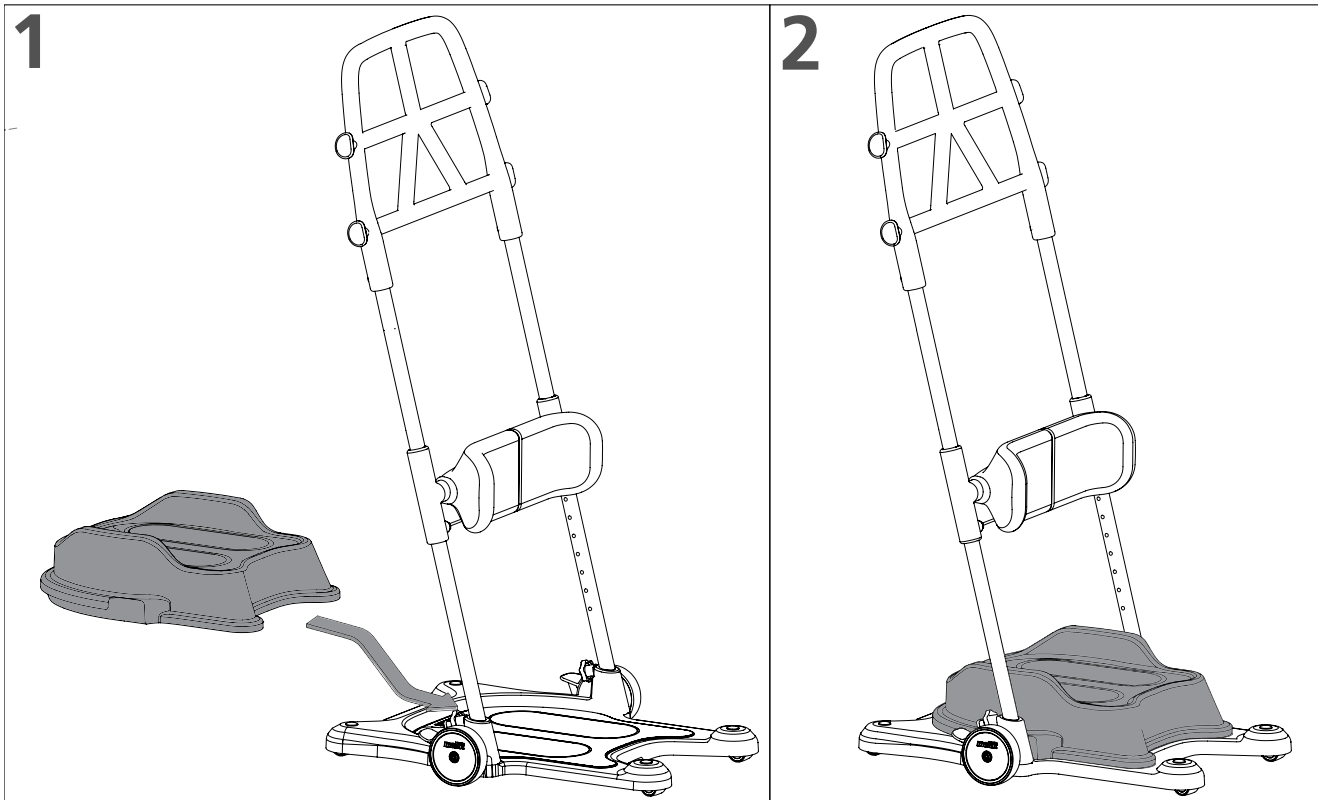- 1 Slide Raiser Pro Block in from assistant side onto the shaped recess on the footplate.
- 2 Make sure Raiser Pro Block is fitted correctly, and safe to use.

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Etac A/S